













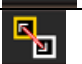



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
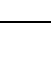




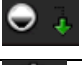







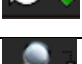


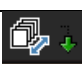


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
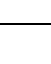


Tool Button	Function	Keyboard Shortcut	Tool Button	Function	Keyboard Shortcut
	Allows more options.	/		Show or Hide Reference Lines.	R
	View past/current reports for patient.	V		Auto Linked Stacking of images in same plane.	L
	Stack Through - the Images in a series.	S		Text Overlay – add/remove text overlay on image.	O
	Magnification option.	G		Select # of screens for layout.	Number Keys 1-9
	Contrast window level - Left click and drag to change contrast.	W		Select layout images within a series window.	CTRL + 1-9
	Pan - select then left click and drag to pan.	P		Add/remove the study list and thumbnail strip.	T
	Rotate image - Click and drag to rotate.	Shift + R		Play images in video format (i.e ultrasound imaging)	CTRL + I
	Zoom in/out.	Z		Image inversion	I
	Drag and Swap images to new position.	D		Reset images within selected viewport except measurements or text.	CTRL + Backspace
	Linear Measurement – picture changes based on selected tool.	M		3D cursor – Left click and drag	C

Tool Button	Function	Keyboard Shortcut	Tool Button	Function	Keyboard Shortcut
	Allows more options.	/		Show or Hide Reference Lines.	R
	View past/current reports for patient.	V		Auto Linked Stacking of images in same plane.	L
	Stack Through - the Images in a series.	S		Text Overlay – add/remove text overlay on image.	O
	Magnification option.	G		Select # of screens for layout.	Number Keys 1-9
	Contrast window level - Left click and drag to change contrast.	W		Select layout images within a series window.	CTRL + 1-9
	Pan - select then left click and drag to pan.	P		Add/remove the study list and thumbnail strip.	T
	Rotate image - Click and drag to rotate.	Shift + R		Play images in video format (i.e ultrasound imaging)	CTRL + I
	Zoom in/out.	Z		Image inversion	I
	Drag and Swap images to new position.	D		Reset images within selected viewport except measurements or text.	CTRL + Backspace
	Linear Measurement – picture changes based on selected tool.	M		3D cursor – Left click and drag	C

IntelViewer Quick Tips


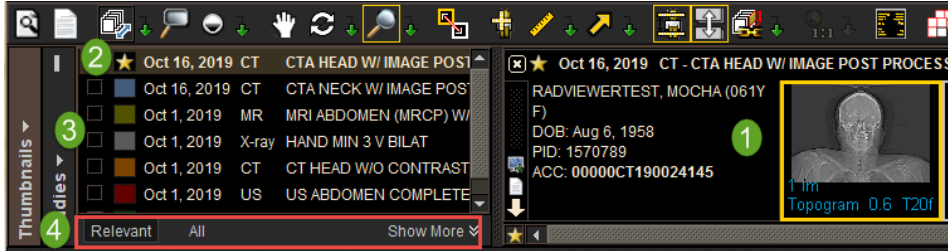
- IntelViewer opens for review of radiology results.
- Access image by selecting Image Viewer icon. 

Image viewing window



Relevant Viewing Window



1. Image will appear in thumbnails. Drag to viewport for viewing.
2. ★ Indicates current study viewing.
3. Colors will indicate previous studies.
4. Select **Relevant** or **All** filter for other studies.

Mouse and Keyboard Navigation

- **Left click and drag** the desired thumbnail to the desired image viewport. The active viewport has a yellow border surrounding it.
- Use **Right and left arrow keys** to navigate between thumbnails.
- Navigate a series of images:
 - **Center scroll**
 - **Up and down arrow keys**
 - **Viewport scroll bar**
- **Right click** on image in viewport to access the radial tool menu.

*****Radial tools will vary depending on the study you are viewing.**




IntelViewer Quick Tips

- IntelViewer opens for review of radiology results.
- Access image by **selecting** Image Viewer icon.

Image viewing window



Relevant Viewing Window

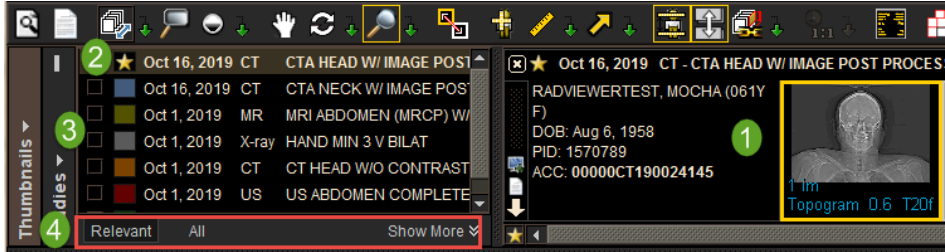

1. Image will appear in thumbnails. Drag to viewport for viewing.
2. ★ Indicates current study viewing.
3. Colors will indicate previous studies.
4. **Select Relevant or All** filter for other studies.

Mouse and Keyboard Navigation

- **Left click and drag** the desired thumbnail to the desired image viewport. The active viewport has a yellow border surrounding it.
- **Use Right and left arrow keys** to navigate between thumbnails.
- Navigate a series of images:
 - **Center scroll**
 - **Up and down arrow keys**
 - **Viewport scroll bar**
- **Right click** on image in viewport to access the radial tool menu.

***Radial tools will vary depending on the study you are viewing.

